



Inlays/Onlays

INLAY PREPARATION


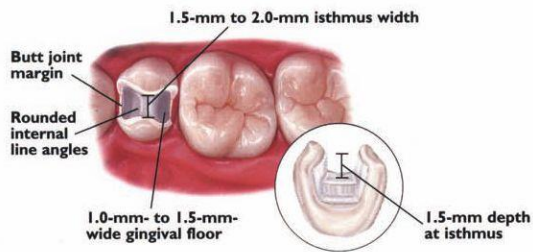


ONLAY PREPARATION

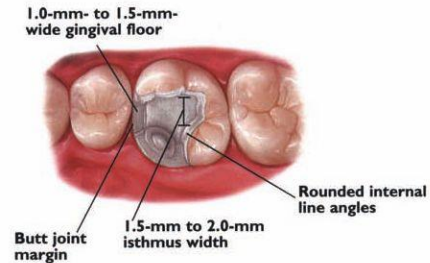


3-Unit Bridge Restorations

3-UNIT BRIDGE PREPARATION

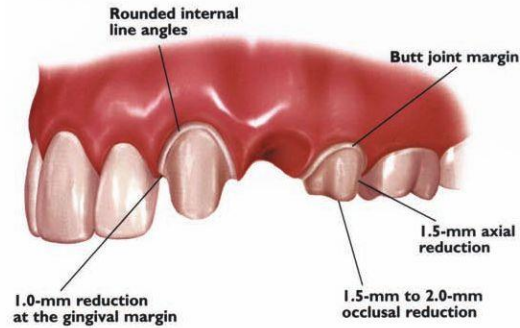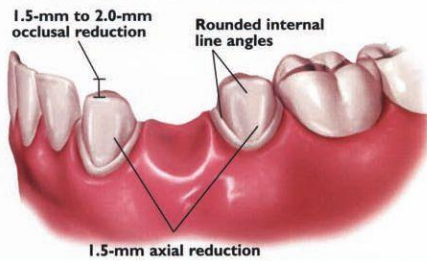


Full-Coverage Restorations

POSTERIOR CROWN PREPARATION

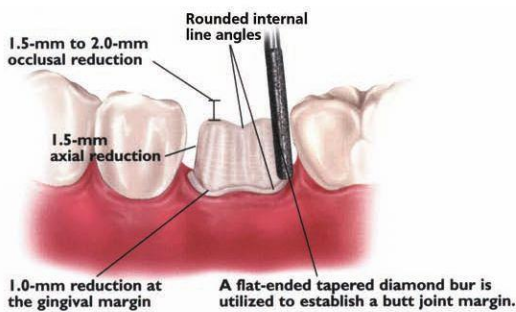

CONVENTIONAL CEMENTATION PREPARATION

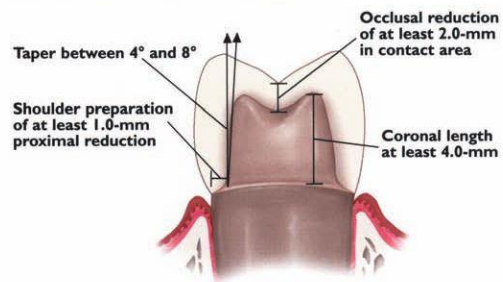

When layered or pressed ceramic margins are preferred in conjunction with a zirconia framework, enhanced gingival esthetics can be achieved with a definitive 90 degree shoulder preparation.

www.ivoclarvivadent.com

For information on these or any other Ivoclar Vivadent products, please call us toll free at 1-800-533-6825 in the U.S., 1-800-263-8182 in Canada. www.ivoclarvivadent.us ©2006 Ivoclar Vivadent, Inc. IPS e.max and IPS Empress are registered trademarks of Ivoclar Vivadent, Inc. Rev.1 7/2008 #604217 All illustrations ©2003 Montage Media Corporation

100% CUSTOMER SATISFACTION
GUARANTEED!

All-Ceramic

Chairside Preparation Guide for IPS Empress® and IPS e.max®

For IPS e.max® restorations

	IPS e.max Ceram/ZirPress	IPS e.max ZirCAD	IPS e.max CAD/Press
Anterior			
Crown		✓	✓
Veneer	✓		
Bridge		✓	✓
Posterior			
Crown		✓	✓
Bridge		✓	


For IPS Empress® restorations

	IPS Empress	
Anterior		
Crown	✓	
Veneer	✓	
Bridge		
Posterior		
Crown	✓	
Bridge		
Inlay/Onlay	✓	

Anterior Chairside Preparation Guide


Full-Coverage Restorations

ANTERIOR CROWN PREPARATION

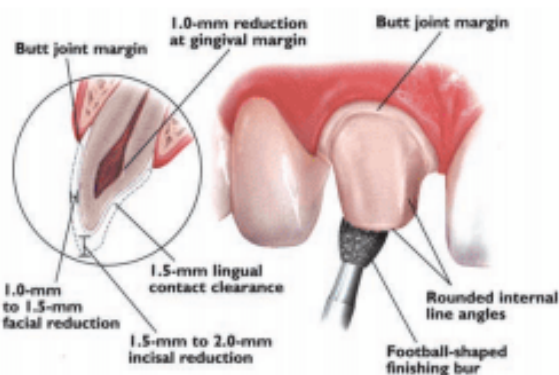

3-Unit Bridge Restorations


3-UNIT BRIDGE PREPARATION

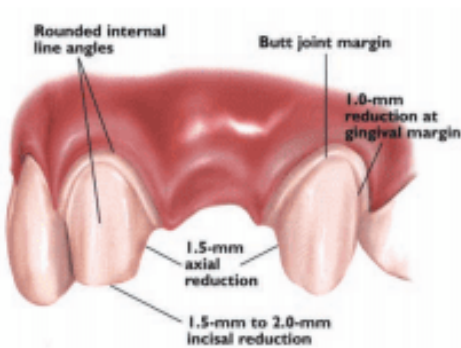

Veneers


VENEER PREPARATION

A medium grit, round-ended, diamond bur is used to remove a uniform thickness of facial enamel by joining the depth-cut grooves.

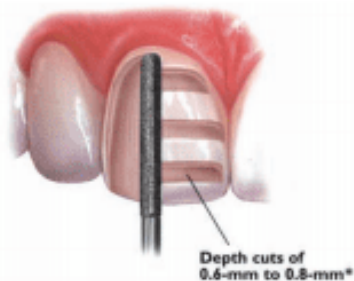


INCISAL PREPARATION

The diamond bur is angled to bevel back the incisal edge.

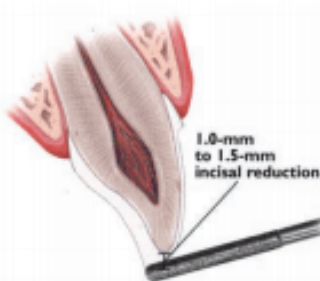

CHAMFER MARGINS

Correct preparation of the chamfer margins interproximally allows the appropriate bulk of porcelain.

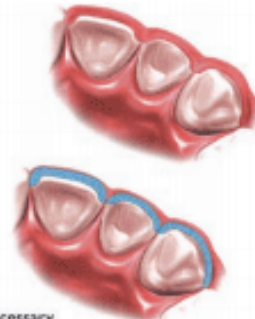

*Please note: For additional masking capabilities and/or layering techniques, further reduction may be necessary. Contact your laboratory for further information.